Rinn® XCP-DS Fit® with XCP-ORA®

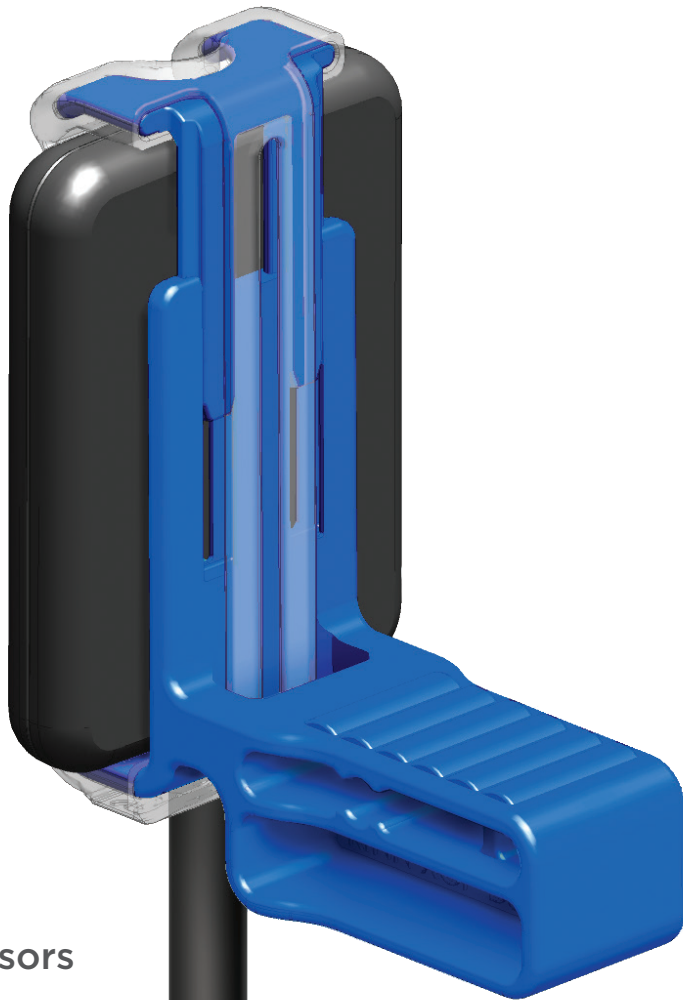
Universal Sensor Holder

Universal-Sensorhalter

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Porta-sensore universale

Sujetador de sensor universal

Fits Size 0 through Size 2 Sensors

Step-by-Step Instructions:

See illustrations on reverse side.

- 1 Place a protective cover over the sensor.
- 2 Load sensor into appropriate biteblock for the area to be imaged by holding sensor firmly and pressing edge of sensor against the movable clip.
- 3 Insert opposite sensor edge and release clip.
4. Insert positioning arm into the appropriate color-coded channel of the aiming ring and attach biteblock according to Positioning Guide.
5. Correctly assembled, the sensor should be centered when viewed through the aiming ring.
6. Position holder in patient's mouth and slide aiming ring close to patient's face.
7. Align x-ray tube to aiming ring.
8. Take exposure.
9. Change as needed for the next desired image.

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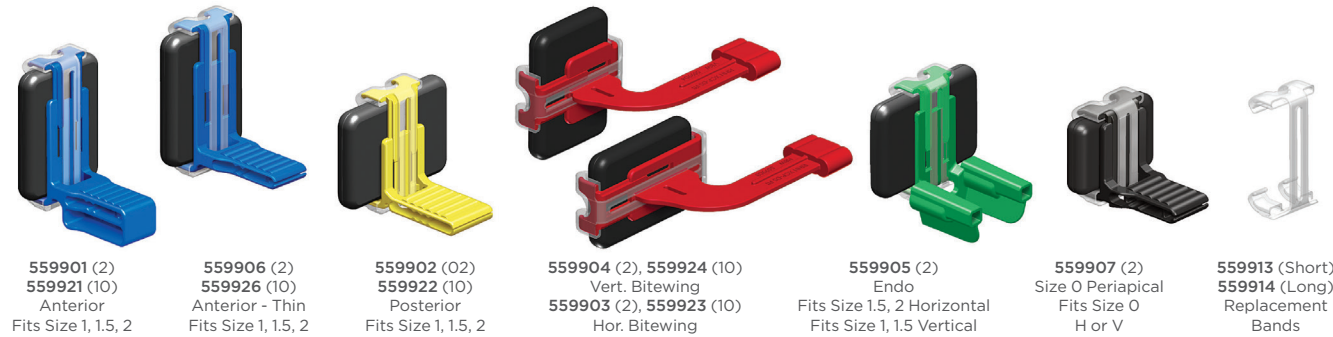
ENGLISH

Caution

U.S. Federal laws restrict this device to sale to or on the order of a licensed dental professional.

Indications for use

XCP-DS FIT holders are indicated for holding digital sensors for intraoral dental radiographs. XCP and XCP-ORA arms and rings are indicated for positioning and aligning intraoral dental radiographs.

Contraindications

None known.

Warnings

Sterilize before use. DO NOT use phenol-based glutaraldehyde or ultrasonic cleaners. DO NOT chemically, dry-heat sterilize or cold sterilize. Only sterilize with steam autoclaving according to cleaning and sterilization instruction. It is the user's responsibility to qualify alternative methods of sterilization or disinfection. Failure to follow properly validated sterilization processes and approved aseptic techniques may result in cross contamination.

Precautions

Follow sterilization instructions carefully to prevent damage to plastic pieces during sterilization. Use sensor holders with a sensor cover. Do not pull on sensor cord during use.

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Reacciones adversas

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ENGLISH

Reprocessing: Disassemble instrument(s) and separate all parts prior to reprocessing.

Manual Cleaning and Disinfection:

1. Rinse components under running tap water using a soft-bristled brush to remove gross soil using an enzymatic detergent per manufacturer's recommendation, paying special attention to any cavities or holes.
2. Rinse under running tap water and dry with cloth or disposable wipe.
3. After thoroughly cleaning, wipe all surfaces with a disposable wipe in combination with a bactericidal, virucidal, and fungicidal instrument disinfection solution (such as VoloWipe) following manufacturer's instructions. Pay special attention to any cavities or holes.
4. Rinse components by wiping them with a cloth soaked in pure water or by running under running tap water.

Automated Cleaning and Disinfection: As an alternative to manual cleaning and disinfection, the device can also be processed in a washer-disinfector. Use only properly maintained, calibrated, and approved washer-disinfectors according to ISO 15883-1.

1. Place devices in washer-disinfector allowing water and detergent to wet all surfaces and to drain off afterwards during washer-disinfector program following manufacturer's instructions. Run a washer-disinfector program with A0 value ≥ 600.

Sterilization:

1. Place components in sterilization pouches, and place in the middle tray of the autoclave, away from autoclave walls and heating element. Plastic parts must be in a separate pouch from metal parts to avoid melting or warping.
2. Cycle steam autoclave at 270°F (132°C) for 10 minutes at 30 PSI (206.8 kPa). Do not exceed 272°F (134°C).
3. To maintain sterility, instruments should remain bagged until ready for use.

Storage: Store at room temperature (18-25°C / 65-77°F).

1. Place the components in the cleaning and disinfection solution, and place in the middle tray of the autoclave, away from autoclave walls and heating element. Plastic parts must be in a separate pouch from metal parts to avoid melting or warping.
2. Cycle steam autoclave at 270°F (132°C) for 10 minutes at 30 PSI (206.8 kPa). Do not exceed 272°F (134°C).
3. To maintain sterility, instruments should remain bagged until ready for use.

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XCP-DS FIT® Kits with XCP-ORA® Arms & Rings

559900 XCP-DS FIT® Hygiene Kit

Contains: XCP-ORA® Arm and Ring; 8 XCP-DS FIT® Biteblocks (2 each: Anterior, Posterior, Horizontal Biting, Vertical Biting); 50 Universal Sensor Covers

559908 XCP-DS FIT® Endodontic Kit

Contains: 1 Endo Ring, 1 Endo Arm, 2 XCP-DS FIT® Endo Biteblocks

559909 XCP-DS FIT® Hygiene Kit plus Endodontic Holders

Contains: XCP-ORA® Arm and Ring; Endo Arm and Ring; 10 XCP-DS FIT® Biteblocks (2 each: Anterior, Posterior, Horizontal Biting, Vertical Biting, Endo); 50 Universal Sensor Covers

XCP-DS FIT® Kit with XCP® Arms & Rings

559910 XCP-DS FIT® Hygiene Kit with XCP® Arms and Rings

Contains: 4 XCP® Arms and Rings (1 Anterior, 2 Posterior, 1 Biting); 8 XCP-DS FIT® Biteblocks (1 Anterior, 1 Anterior - Thin, 2 Posterior, 2 Horizontal Biting, 2 Vertical Biting); 50 Universal Sensor Covers

Rinn® Digital Sensor Covers

550200 Rinn® Large Digital Sensor Covers, 200-pk

550500 Rinn® Universal Digital Sensor Covers, 500-pk

XCP-DS FIT® Biteblocks

2-packs

559901 Anterior

559906 Anterior - Thin

559902 Posterior

559903 Horizontal Biting

559904 Vertical Biting

559905 Endo

559907 Size 0 Periapical

10-packs

559921 Anterior

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559922 Posterior

559923 Horizontal Biting

559924 Vertical Biting

XCP-DS FIT® Replacement Bands

6 per pack

559913 Short (for Posterior, Vertical Biting, Endo and Size 0)

559914 Long (for Anterior, Anterior - Thin, and Horizontal Biting)

Rinn® XCP-DS Fit® with XCP-ORA®

Universal Sensor Holder

Universal-Sensorhalter
Porte-capteur universel
Porta-sensore universale
Sujetador de sensor universal

Positioning Guide

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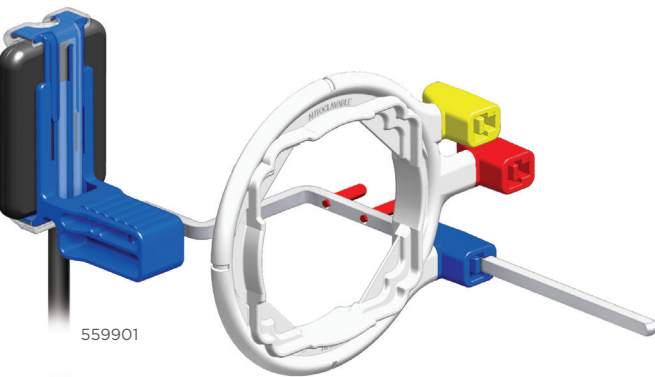
Dentsply RINN
1301 Smile Way
York, PA 17404, USA


EC REP
DENTSPLY Limited
Building 3, The Heights
Weybridge, Surrey KT13 0NY UK

Anterior

Frontzahn / Antérieur / Anteriore / Anteriores

559901

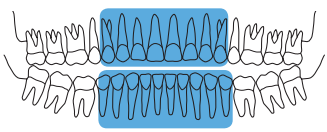Always use a sensor cover to protect sensor.

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